KO

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UK

## SALT BREATHING

Salt inhalation therapy is traditionally used in the treatment of respiratory diseases such as asthma, respiratory allergies, etc. In the patented Polar salt breathing device, an air current causes the salt grains (NaCl) to rub against each other and form a fine salt dust.

Through breathing, the salt particles are transported to the outermost parts of the lungs and an organic purification reaction occurs.

The best treatment results are achieved in 10-30 treatment periods, 1-3 times a day.

A course of treatment lasts 15-60 minutes depending on the age and severity of the disease.

Salt dust can also be blown for example into the bedroom air in the evening. The useful electrically charged particles remain in the air overnight.

### Prior use:

Allow the new Polar Salt Breathing Device to operate without salt for 2-3 hours to remove any packaging odors which may exist.

### Directions for use:

Open the generator cover.

Place 1-2 teaspoons of pure and dry treatment salt (NaCl) into the cylinder.

Place the generator cover back in place.

Set the breathing funnel on patient's chest.

Turn the power switch on.

The consumption of salt during one treatment is 0.1-0.5 g. After treatment, the salt should be removed from the cylinder. The remaining salt in the capsule can be used again but, if necessary, should be dried before re-use. This can be done by heating.

The salt used in the Polar Salt Breathing Device is pure dried medical salt (NaCl), thus achieving the correct dosage and assuring functionality of the device.

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